

Disinfection, Cleaning, Hygiene

# GREEN & CLEAN

Hands

Surfaces

Special Areas

Instruments



 **METASYS**



# GREEN&CLEAN in the dental practice

Microorganisms are part of our daily lives. The risk of infection is usually not considered particularly high. Not so in medical practices - the risk of infection is much higher here. The frequent change of patients and the nature of the work itself require consequent hygiene measures. The responsibility of the physician towards his staff, the patient and himself calls for the use of highly effective cleaning and disinfecting agents.

With our GREEN&CLEAN product range, METASYS offers you a comprehensive solution to meet all hygiene requirements and guidelines. Advanced active and conditioning agents ensure absolute safety and infection protection. CE marks and VAH listings vouch for the products' high quality standards.

Ensure optimal hygiene in all areas of the dental practice with GREEN&CLEAN!

## GREEN&CLEAN HL | GREEN&CLEAN HD N GREEN&CLEAN HC

GREEN&CLEAN HL - Nourishing, antiseptic hand wash lotion

GREEN&CLEAN HD N - Disinfectant for hygienic hand disinfection and surgical hand and forearm disinfection

GREEN&CLEAN HC - Caring hand cream



## GREEN&CLEAN WD


Alcoholic surface disinfection for small surfaces, such as hand and contra-angle pieces


## GREEN&CLEAN IK

Disinfection of spray water and spray air channels of handpieces, contra-angles and turbines

## GREEN&CLEAN ID N

Disinfecting preliminary cleaning of invasive and non-invasive instruments

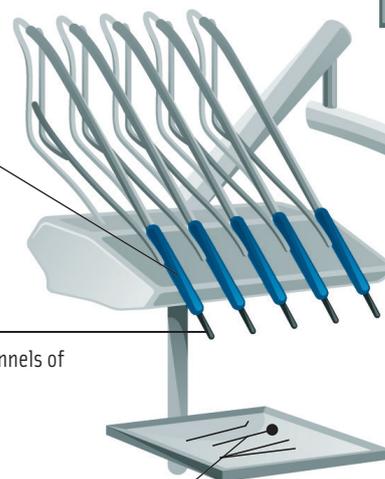

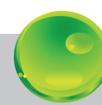

## **GREEN&CLEAN WK | GREEN&CLEAN BR**

GREEN&CLEAN WK - Decontamination and protection against limescale and biofilm in water pipes  
GREEN&CLEAN BR - Biofilm removal

## **GREEN&CLEAN SD | GREEN&CLEAN MT**

Alcoholic surface disinfection for surfaces insensitive to alcohol, such as floating tables and shelves

## **GREEN&CLEAN M2 | GREEN&CLEAN CL N**

GREEN&CLEAN M2 - Cleaning, disinfecting and deodorizing of suction systems and amalgam separators  
GREEN&CLEAN CL N - Suction system cleaning

## **GREEN&CLEAN SK | GREEN&CLEAN MK**

Surface disinfection without alcohol for alcohol-sensitive surfaces of the dental unit: Unit handles, leather/synthetic leather covers of the dental unit, hoses

## **GREEN&CLEAN MB | GREEN&CLEAN AD**

GREEN&CLEAN MB - Spittoon bowl cleaner  
GREEN&CLEAN AD - Disinfection of impressions



In everyday work life you constantly come into contact with transient germs. National and international studies prove that hands are THE transmission path for pathogens. Hand disinfection is therefore considered crucial for the prevention of nosocomial infections. In Europe alone, up to 5 million patients in hospitals and other healthcare facilities are infected annually with dangerous pathogens. A major reason for this is inadequate or even lack of hygiene. Comprehensive hand hygiene can reduce the rate of "acquired" infections up to 40%.

Proper hand hygiene helps to prevent skin problems and maintain the natural protective function of your skin! Hence: Disinfection instead of washing (except in case of contamination)! Ensure that your hand disinfectant does not leave your skin dry, brittle and cracked. Always allow your hands to dry completely before putting on gloves.

With our GREEN&CLEAN hand product line - wash lotions, disinfectant and hand cream - we offer you a simple solution to protect your staff and patients from dangerous infectious diseases while ensuring gentle skin care with nourishing ingredients.

## Dispensing systems

### Dispensing systems for hand disinfectants and liquid soaps

**META TOUCH** wall dispenser for 1000 ml euro bottles  
dishwasher safe and autoclavable up to 134°C  
dosage adjustable to 0.75 ml / 1.0 ml / 1.5 ml per stroke  
housing made of sturdy aluminum  
easy pump removal from the front

**META TOUCHLESS** dispensing system for contactless, sensor-controlled dosing for 500 ml euro bottles  
dosage: 1.5 ml per stroke  
flexible positioning: As tabletop dispenser or wall-mounted  
removable drip tray

#### Order numbers

40200105 META TOUCH, wall dispenser for 1000 ml euro bottles  
40200106 META TOUCHLESS, dispensing system for 500 ml euro bottles



## GREEN&CLEAN HL Hygienic hand cleaning

### Nourishing hand wash lotion with broad antimicrobial action

**Characteristics** broad antiseptic action  
alkali-free, pH-neutral  
supports the natural acid protection of the skin  
prevents skin irritations  
with care substance allantoin

#### Order numbers

60030151 Euro bottle 1000 ml  
60030051 Intro kit, 5 x 750 ml + dispensing bottle  
60030052 Refill kit 1, 6 x 750 ml  
60030053 Refill kit 2, 1 x canister 5 l  
40200004 Water tap 5 l canister



## GREEN&CLEAN HD N\*

### Hand disinfection

**FULLY VIRUCIDAL**  
in only 60 sec.



**Nourishing rub-in preparation for the hygienic hand disinfection and surgical hand and forearm disinfection**

**Characteristics** high-quality alcohols prevent the dehydration of the skin  
sustains the skin's natural pH-value  
pleasant pine scent  
VAH listed  
meets the following EN standards:  
> EN 1500 hygienic hand disinfection  
> EN 12791 surgical hand disinfection  
> EN 14476 virucidal effectiveness  
> EN 14348 mycobactericidal effectiveness

**Active agent** Propan-1-ol, Ethanol

**Spectrum of efficacy** bactericidal, fungicidal (C. albicans), tuberculocidal, fully virucidal

**Action time**

<b>hygienic hand disinfection</b>	rub 3 ml of GREEN&CLEAN HD N on hands for 30 sec
<b>surgical hand disinfection</b>	rub 10 ml of GREEN&CLEAN HD N on hands and forearms for 1.5 min

**Order numbers**

60030168	GREEN&CLEAN HD N, 6 x 1000 ml euro bottles
60030169	GREEN&CLEAN HD N, 6 x 500 ml euro bottles

#### Application



\*Use biocides safely. Always read the label and product information before use.

## GREEN&CLEAN HC

### Hand care



**Caring hand cream with shea butter, ectoin and allantoin**

**Characteristics** suitable for sensitive skin  
nourishes the skin and provides moisture for supple and relaxed skin  
counteracts skin cracks and brittleness  
absorbs quickly

**Order numbers**

60030122-1	Start, 1 x 100 ml dispenser
60030120-1	Refill kit, 6 x 100 ml dispensers

# SURFACES

After each treatment, surfaces which are close to the patient and have been contaminated through direct contact or aerosols need to be cleaned and disinfected. For aerosols, a dispersal radius of 2 meters must be considered.

Also, all areas which are farther away from the patient but have been visibly contaminated must be disinfected immediately. Once every working day, all other functional areas in the clinically used practice area must be treated with a surface disinfectant. For reasons of occupational safety, wipe disinfection is generally preferable to spray disinfection.


## Non-alcoholic surface disinfection

### Especially suitable for

- > leather and synthetic leather covers of the treatment chair
- > handles of the dental unit, e.g. made of rubber
- > hoses of the dental unit
- > acrylic or plexiglass (lamps)

### Advantages of non-alcoholic surface disinfection

- > excellent material compatibility
- > cost savings through longer service life
- > can be used in the entire practice

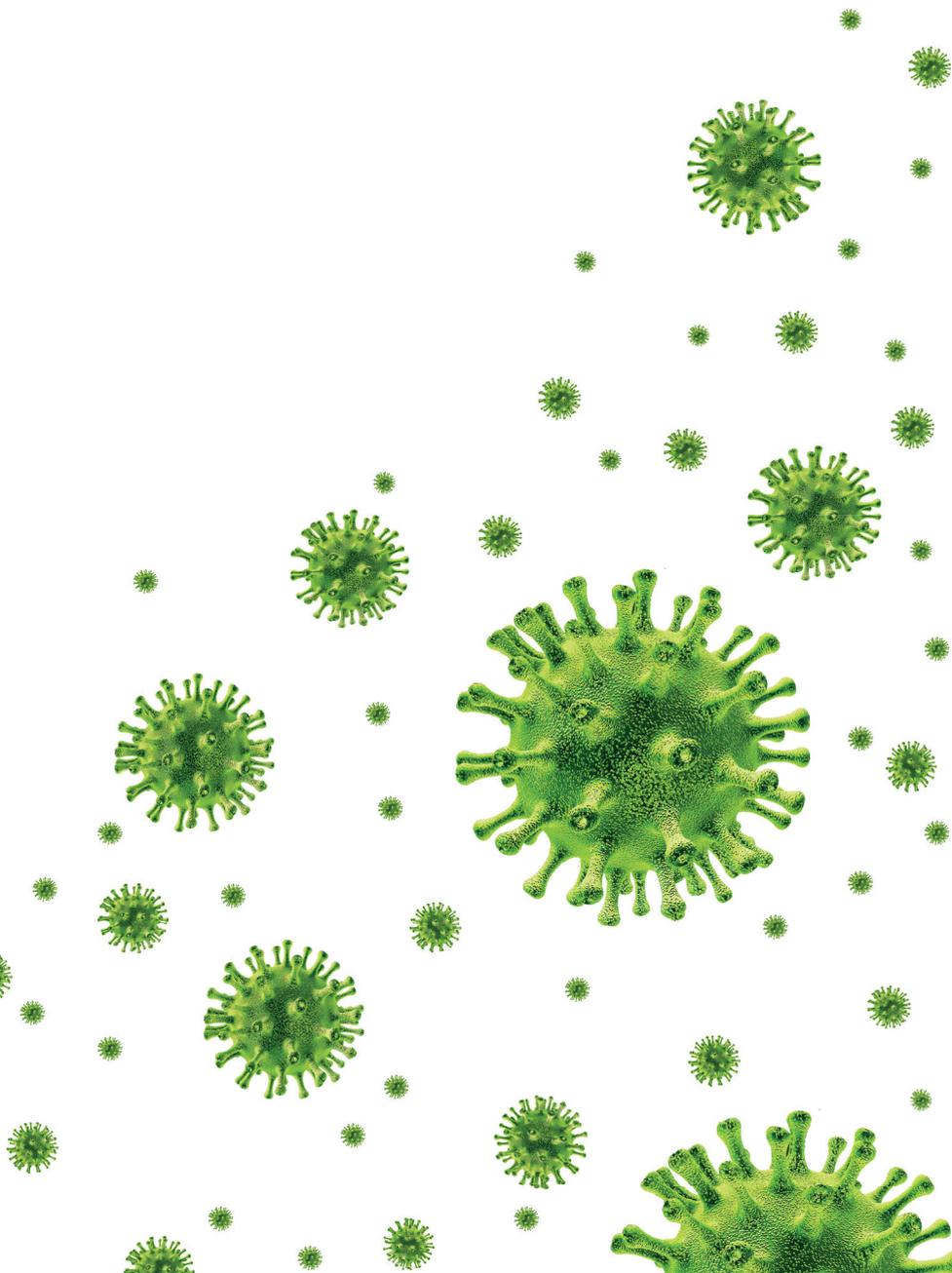

# GREEN&CLEAN SK & MK Spray and wipe disinfection for large areas

**WITHOUT ALCOHOL**



Non-alcoholic spray and wipe disinfection for alcohol-sensitive surfaces of medical devices (e.g. unit handles, leather/synthetic leather covers of the dental unit)

**GREEN&CLEAN SK** Disinfecting foam without alcohol for the disinfection of surfaces of medical devices

**GREEN&CLEAN MK** Extra large wipes (290 x 300 mm) Disinfecting solution without alcohol for the disinfection of large surfaces of medical devices

**Characteristics**

- alcohol-free
- short action time
- ready-to-use solution
- long-lasting effect against re-establishment of airborne bacteria and viruses
- particularly well suited for use on alcohol-sensitive surfaces of medical devices
- aldehyde-, phenol- and phosphate-free
- CE 0044

**Modern active agent combination**

Quaternary ammonium compounds

<b>Spectrum of efficacy</b>	bactericidal**	1 min
<b>/ Action time</b>	fungicidal (C. albicans)**	1 min
**under low load	HBV/HIV/HCV	1 min

**Expert opinions**

Prof. Dr. med. G. Frösner, Munich  
 Prof. Dr. med. E. Marth, Graz  
 Prof. Dr. med. R. Schubert, Frankfurt/Main  
 Dr. med. J. Steinmann, Bremen


**Order numbers GREEN&CLEAN SK**

60030110	Start, 1 x 750 ml spray bottle with foamer
60030111	Intro kit, 6 x 750 ml spray bottles + 2 x foamer
60030112	Refill kit, canister 5 l
40200004	Water tap 5 l canister

**Order numbers GREEN&CLEAN MK**

60030130	Start 1, 1 x dispenser with 70 wipes + 750 ml bottle GREEN&CLEAN SK
60030131	Start 2, 1 x dispenser with 70 wipes + 5 x 70 wipes + 1 x 5 l GREEN&CLEAN SK canister
60030095	MK/MT Refill kit 1, 2 x 70 wipes
60030092	MK/MT Refill kit 2, 1 x dispenser with 70 wipes + 5 x 70 wipes

**Application**

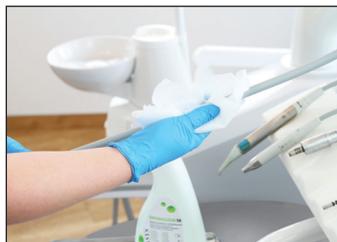

## Alcoholic surface disinfection

### Especially suitable for

- > alcohol-insensitive surfaces of the dental unit, such as floating tables and shelves
- > outer surfaces of handpieces and contra-angles

### Advantages of alcoholic surface disinfection

- > broad spectrum of efficacy
- > short action time

## GREEN&CLEAN WD Wipe disinfection for small areas

Alcoholic disinfecting wipes from tearproof fleece for disinfecting small surfaces of medical devices (e.g. outer surfaces of straight and contra-angle handpieces)

<b>Characteristics</b>	short action time
	long-lasting effect against re-establishment of airborne bacteria and viruses
	with hygienic safety sealing
	formaldehyde-free
	CE 0044

### Modern active agent combination

Alcohols, quaternary ammonium compounds, glyoxal

<b>Spectrum of efficacy</b>	bactericidal*	1 min
<b>/ Action time</b>	tuberculocidal*	5 min
*under high load	fungicidal ( <i>C. albicans</i> )*	1 min
	limited virucidal (e.g. HBV/HIV/HCV, HSV-I)	30 sec

### Expert opinions

Prof. Dr. med. L. Döhner, Greifswald  
 Prof. Dr. med. H.-P. Werner, Schwerin  
 Dr. med. J. Steinmann, Bremen  
 Prof. Dr. med. G. Frösner, Munich

### Order numbers

60030018	Start, 1 x box with 120 wipes
60030011	Intro kit, 1 x box with 120 wipes + 3 x refill bags à 120 wipes
60030012	Refill kit, 6 x refill bags à 120 wipes
40200021	GREEN&CLEAN WD box, empty

### Application



# GREEN&CLEAN SD & MT Spray and wipe disinfection for large areas



Alcoholic spray and wipe disinfectant for alcohol-insensitive surfaces of medical devices (e.g. floating tables and shelves)

**GREEN&CLEAN SD** Alcoholic spray disinfection for the disinfection of surfaces of medical devices

**GREEN&CLEAN MT** Extra large wipes (290 x 300 mm)  
Alcoholic disinfecting solution for the disinfection of large surfaces of medical devices

**Characteristics** short action time  
ready-to-use solution  
long-lasting effect against re-establishment of airborne bacteria and viruses  
formaldehyde-free  
drip-free spray head  
CE 0044

### Modern active agent combination

Alcohols, quaternary ammonium compounds, glyoxal

<b>Spectrum of efficacy</b>	bactericidal*	1 min
<b>/ Action time</b>	tuberculocidal*	5 min
	fungicidal ( <i>C. albicans</i> )*	1 min
	limited virucidal (e.g. HBV/HIV/HCV, HSV-I)	30 sec

\*under high load

**Expert opinions** Prof. Dr. med. L. Döhner, Greifswald  
Prof. Dr. med. H.-P. Werner, Schwerin  
Dr. med. J. Steinmann, Bremen  
Prof. Dr. med. G. Frösner, Munich

### Order numbers GREEN&CLEAN SD

60030001	Intro kit, 5 x 750 ml bottles + 2 x spray heads
60030002	Refill kit 1, 6 x 750 ml bottles
60030003	Refill kit 2, 1 x canister 5 l
60300001	Refill kit 3, 2 x 750 ml bottles + spray head
40200004	Water tap 5 l canister

### Order numbers GREEN&CLEAN MT

60030094	Intro kit 2, 1 x dispenser with 70 wipes + 5 x 70 wipes + 1 x 5 l GREEN&CLEAN SD canister
60030095	MK/MT Refill kit 1, 2 x 70 wipes
60030092	MK/MT Refill kit 2, 1 x dispenser with 70 wipes + 5 x 70 wipes

### Application



Especially in the dental practice, close attention should be paid to the cleaning and disinfection of special areas. This applies to amalgam separators and suction systems, as well as spittoon bowls and the disinfection of impressions and prosthetic workpieces.

Above all, water quality is one of the most important hygiene topics. The formation of biofilm in water-bearing lines represents a significant health risk. Contaminated and polluted process water increases the risk of infection for patients and staff. Continuous decontamination allows for germ reduction and protection against infection by e.g. Legionella.

## GREEN&CLEAN M2 Suction system disinfection

**Dual, actively defoaming concentrate for the daily cleaning, disinfection and deodorization of suction systems and amalgam separators**

<b>Characteristics</b>	dual system (red/green) active against resistance excellent active defoaming easy handling through well-designed dispenser high environmental compatibility good material compatibility enzymatic product with excellent protein dissolving power ideal for all amalgam separators and air-water separators CE 0044
------------------------	--

### Modern active agent combination

Alkylamine, quaternary ammonium compounds, enzymes

<b>Spectrum of efficacy</b>	bactericidal*	15 min (1% solution)
<b>/ Action time</b>	tuberculocidal*	240 min (1% solution)
<small>*under high load</small>	fungicidal (C. albicans)*	15 min (1% solution)
	HBV/HIV/HCV	60 min (1% solution)

<b>Expert opinions</b>	Prof. Dr. med. R. Schubert, Frankfurt/Main Dr. med. J. Steinmann, Bremen Prof. Dr. med. H.-P. Werner, Schwerin
------------------------	--

### Order numbers

60010201	Intro kit, 2 x 500 ml bottles (1 x red, 1 x green) + dispenser
60010202	Refill kit 1, 4 x 500 ml bottles (2 x red, 2 x green)
60010203	Refill kit 2, 2 x 500 ml bottles (1 x red, 1 x green)
60010204	GREEN&CLEAN M2 red, 25 x 500 ml bottles
60010205	GREEN&CLEAN M2 green, 25 x 500 ml bottles
40200001-BL	Dispenser for GREEN&CLEAN M2



### Application



1. Dose 6 ml (press 2x)



2. Dilute with water, 600 ml



3. Suction, 200 ml each



4. Empty the remaining solution (200 ml) into the spittoon bowl and allow to act

## GREEN&CLEAN H1



### Cleaning of the H1 Hygiene System

Decontamination concentrate for the automatic METASYS H1 Hygiene System

**Characteristics** actively defoaming  
optimal cleaning effect through enzymes  
CE

**Modern active agent combination**  
Quaternary ammonium compounds, special tensides, enzymes

**Order number**  
60010021 Refill kit, 4 x 500 ml pouches

## GREEN&CLEAN CL N



### Suction system cleaning

Concentrate for the daily cleaning of suction systems

**Characteristics** better defoaming thanks to modern active agent combination  
excellent protein dissolving power thanks to a special enzyme mixture  
pleasant scent in the treatment rooms  
simple dosing thanks to the double-neck bottle  
pH-neutral user solution

**Modern active agent combination**  
Subtilisin, alkyl polyethylene glycol ether

**Order numbers**  
60010305 GREEN&CLEAN CL N, 2 x 1000 ml bottles  
40200023 Mixing container for GREEN&CLEAN CL N

## GREEN&CLEAN MB



### Spittoon bowl cleaning

Practical dispenser with integrated brush based on citric acid and enzymes for an optimal cleaning of the spittoon bowl

**Characteristics** dissolves limescale and color sedimentation  
excellent protein dissolving power through enzymatic ingredients  
with special defoamer

**Modern active agent combination**  
Quaternary ammonium compounds, tensides, enzymes

**Order numbers**  
60020005 Start, 1 x 180 ml dispensing bottle with brush  
60020001 Intro kit, 2 x 500 ml + 2 dispensing bottles + 2 brushes  
60020002 Refill kit 1, 4 x 500 ml + 4 brushes  
60020004 Refill kit 2, 4 x 500 ml  
60020008 Replacement brushes, 4 pcs.  
60020009 Dispensing bottle with brush, empty

# SPECIAL AREAS

## GREEN&CLEAN WK Water decontamination and biofilm prevention

2% hydrogen peroxide solution for the decontamination of process water and the prevention of limescale deposits in the pipe systems of dental units

**Characteristics** broad spectrum of efficacy  
prevention of biofilm  
with long-term effect  
limescale inhibitor prevents calcification  
protects hand and angle pieces  
also suitable for bottle systems  
CE 0044

**Active agent** Hydrogen peroxide

### Order numbers

60040100-1 Refill kit 1, 4 x 750 ml bottles  
60040101 Refill kit 2, 6 x 1000 ml bottles  
40200020 Dosing kit for bottle systems (for 750 ml bottle)

### GREEN&CLEAN WK Test strips

Test strips used to conduct a visual inspection of the functionality and the effectiveness of the water decontamination system

**Characteristics** the color scale on the test strip indicates the concentration of GREEN&CLEAN WK in the process water  
WEK Light: 1x weekly inspection required  
WEK: 1x weekly inspection recommended

### Order numbers

40050502 GREEN&CLEAN WK test strips 5 pcs.  
40050503 GREEN&CLEAN WK test strips 25 pcs.  
40050504 GREEN&CLEAN WK test strips 50 pcs.



## METASAN® P Water decontamination and biofilm prevention

1.41% solution for the disinfection of the process water in dental treatment units, based on hydrogen peroxide

**Characteristics** continuous decontamination of process water and the water-bearing lines  
high material compatibility  
broad spectrum of efficacy  
prevention of biofilm  
with long-term effect  
for Dentsply Sirona treatment units  
CE 0044

**Active agent** Hydrogen peroxide

### Order number

60040021 METASAN® P 6 x 1000 ml bottles



## GREEN&CLEAN BR



### Biofilm removal

4% hydrogen peroxide solution for the removal of biofilm in the water-bearing pipes of the dental unit with the METASYS BR biofilm removing system or bottle systems

**Characteristics** effective biofilm removal in only 30 min  
simple application through colored solution  
recommended for the refurbishment of dental units as well as before the installation of a water decontamination system CE 0044

**Active agent** Hydrogen peroxide

**Order number**  
60040300 Biofilm removing solution, 2 x 1000 ml bottles

## GREEN&CLEAN AD



### Disinfection of impressions

Ready-to-use solution for the sustained immersion disinfection of all prosthetics, worn dentures and dental impressions

**Characteristics** short action time  
form stability certificate (Fresenius Institute)  
can be used for alginates, silicones, polyether rubber and impression trays  
no impairment of surface quality or dimensional stability of the impressions  
excellent cleaning power  
high material compatibility  
formaldehyde- and phenol-free  
standing time: 7 days  
CE 0044

**Modern active agent combination**  
Quaternary ammonium compounds, alkylamine, tensides

<b>Spectrum of efficacy / Action time</b>	bactericidal*	5 min
	tuberculocidal*	5 min
	fungicidal ( <i>C. albicans</i> )*	5 min
	limited virucidal (e.g. HBV/HIV/HCV, HSV-1)	5 min

\*under high load

**Expert opinions** Prof. Dr. med. R. Schubert, Frankfurt/Main  
Prof. Dr. med. E. Marth, Graz  
Dr. med. J. Steinmann, Bremen

**Order numbers**  
60030074 1 x canister 5 l  
40200015 Disinfection tray 1,25 l with cover and strainer  
40200004 Water tap 5 l canister

#### Application



1. Prepare disinfection tray




2. Rinse impressions thoroughly with water



3. Immerse impressions in the disinfection tray



4. Observe action time

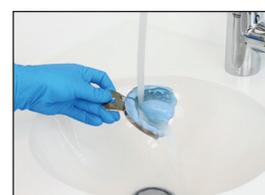

5. Rinse impressions thoroughly

Proper hygienic reprocessing of transmission instruments is an important part of infection prophylaxis in the dental field. Not only used instruments but also all instruments that have been laid out openly must be considered contaminated and therefore be subjected to hygienic reprocessing without exception. The use of suitable preparations not only allows for effective infection control but also helps to preserve the service life and functionality of expensive equipment.

From the disinfection of handpieces and turbines, to the manual disinfecting preliminary cleaning of invasive and non-invasive instruments, the GREEN&CLEAN product line for instruments by METASYS covers all necessities in terms of dental instruments.

## GREEN&CLEAN IK Disinfection of handpieces, contra-angles and turbines

**Ready-to-use, alcoholic disinfectant spray for the spray water and spray air channels of handpieces, contra-angles and turbines**

<b>Characteristics</b>	short action time
	increased application security through pre-assembled adapter
	one bottle suffices for approx. 300 applications, when spraying for approx. 2 seconds each time
	one universal adapter for all handpieces & contra-angles, as well as turbine couplings and custom adapters for the most common manufacturers
	does not replace oil spray recommended by manufacturer!
	tested according to EN standards
factory release (W&H, Dentsply Sirona)	
CE 0044	

### Modern active agent combination

Alcohols, quaternary ammonium compounds

<b>Spectrum of efficacy</b>	bactericidal*	2 min
<b>/ Action time</b>	tuberculocidal*	2 min
*under high load	fungicidal (C. albicans)*	2 min
	limited virucidal (e.g. HBV/HIV/HCV, HSV-I)	15 sec

### Expert opinions

Prof. Dr. med. M. Rotter, Vienna  
 Dr. med. J. Steinmann, Bremen  
 Prof. Dr. med. H.-P. Werner, Schwerin

### Order numbers

2 x 200 ml spray cans incl. pre-assembled adapter:

60030182	Hand pieces and contra-angles (universal)
60030183	Turbine coupling W&H
60030184	Turbine coupling Bien Air
60030185	Turbine coupling KaVo
60030186	Turbine coupling Dentsply Sirona
60030187	Turbine coupling Castellini
60030188	Turbine coupling NSK
60030189	Hand pieces and contra-angles Dentsply Sirona T1 Classic
60030190	Turbine coupling NSK Quick



# GREEN&CLEAN ID N

## Disinfecting pre-cleaning of instruments



**Highly effective concentrate for the disinfecting preliminary cleaning of invasive and non-invasive instruments**

**Characteristics**

- extraordinary cleaning effect
- suitable for flexible and rigid instruments, as well as heat-sensitive and -insensitive instruments
- 5 l of concentrate result in 500 l of working solution
- prevents corrosion for the protection of your instruments
- suitable for use in the ultrasonic bath
- aldehyde- and phenol-free
- recommendation of the endoscope manufacturer Karl Storz GmbH & Co. KG, Tuttlingen (Germany)
- release and recommendation of NTI-Kahla (Rotary Dental Instruments)
- VAH listed
- standing time: 7 days
- CE 0044

### Modern active agent combination

Quaternary ammonium compounds, alkylamine, corrosion inhibitors

### Spectrum of efficacy / Action time / Dilution

*under high load	5 min	15 min	30 min	60 min
bactericidal*	1.5%	0.5%	0.25%	0.1%
tuberculocidal*	7.5%	2.0%	2.5%	1.0%
fungicidal (C. albicans)*	1.5%	0.5%	0.25%	0.1%
limited virucidal (e.g. HBV/HIV/HCV, HSV-1)			2.0%	

**Expert opinions**

Prof. Dr. med. R. Schubert, Frankfurt/Main  
 Prof. Dr. med. E. Marth, Graz  
 Dr. med. J. Steinmann, Bremen

### Order numbers

60030033	4 x 500 ml bottles + measuring beaker
60030034	1 x canister 5 l
40200003	Measuring beaker for GREEN&CLEAN ID N
40200004	Water tap 5 l canister

### Application



1. Dose and prepare tray



2. Remove coarse contaminants from instruments



3. Immerse instruments in the solution



4. Cover the tray and observe action time



5. Rinse instruments



**Trays and bur disinfection boxes from METASYS guarantee a perfect instrument disinfection**

### Order numbers

40200008	Disinfection tray 4 l with cover and strainer
40200015	Disinfection tray 1,25 l with cover and strainer
40200013	Bur disinfection box 200 ml with cover and strainer

# All information at a glance

## HANDS

		Cleaning	Disinfection	Alcoholic	Ready-to-use	VAH listed	Action time bactericidal	Action time tuberculocidal	Action time fungicidal (C. albicans)	Action time limited virucidal	Action time fully virucidal	Available as
GREEN&CLEAN HL	Hygienic hand cleaning	■			■							750 ml, 1000 ml, 5000 ml
GREEN&CLEAN HD N*	Hand disinfection		■	■	■	■		30 sec		30 sec	60 sec	500 ml, 1000 ml
GREEN&CLEAN HC	Hand care				■							100 ml

## SURFACES

GREEN&CLEAN SK	Spray disinfection	■	■		■		1 min <sup>6</sup>		1 min <sup>6</sup>	1 min <sup>1</sup>		750 ml, 5000 ml
GREEN&CLEAN MK	Wipe disinfection	■	■				1 min <sup>6</sup>		1 min <sup>6</sup>	1 min <sup>1</sup>		750 ml, 5000 ml, 70 wipes
GREEN&CLEAN WD	Wipe disinfection	■	■	■	■		1 min <sup>5</sup>	5 min <sup>5</sup>	1 min <sup>5</sup>	30 sec		120 wipes
GREEN&CLEAN SD	Spray disinfection	■	■	■	■		1 min <sup>5</sup>	5 min <sup>5</sup>	1 min <sup>5</sup>	30 sec		750 ml, 5000 ml
GREEN&CLEAN MT	Wipe disinfection	■	■	■			1 min <sup>5</sup>	5 min <sup>5</sup>	1 min <sup>5</sup>	30 sec		750 ml, 5000 ml, 70 wipes

## SPECIAL AREAS

GREEN&CLEAN M2	Suction system disinfection	■	■				15 min <sup>2,5</sup>	240 min <sup>2,5</sup>	15 min <sup>2,5</sup>	60 min <sup>1,2</sup>		500 ml
GREEN&CLEAN H1	Cleaning of the H1	■			■							500 ml
GREEN&CLEAN CL N	Suction system cleaning	■										1000 ml
GREEN&CLEAN MB	Spittoon bowl cleaning	■			■							180 ml, 500 ml
GREEN&CLEAN WK	Water decontamination and biofilm prevention		■									750 ml, 1000 ml
METASAN® P	Water decontamination and biofilm prevention		■		■							1000 ml
GREEN&CLEAN BR	Biofilm removal	■	■		■							1000 ml
GREEN&CLEAN AD	Disinfection of impressions	■	■		■		5 min <sup>5</sup>	5 min <sup>5</sup>	5 min <sup>5</sup>	5 min		5000 ml

## INSTRUMENTS

GREEN&CLEAN IK	Disinfection of handpieces & contra-angles, turbines	■	■	■	■		2 min <sup>5</sup>	2 min <sup>5</sup>	2 min <sup>5</sup>	15 sec		200 ml
GREEN&CLEAN ID N	Instrument disinfection	■	■			■	5 min <sup>4,5</sup>	15 min <sup>3,5</sup>	5 min <sup>4,5</sup>	30 min <sup>3</sup>		500 ml, 5000 ml

<sup>1</sup> only HBV/HIV/HCV, <sup>2</sup> 1.0% solution, <sup>3</sup> 2.0% solution, <sup>4</sup> 1.5% solution, <sup>5</sup> under high load, <sup>6</sup> under low load  
\*Use biocides safely. Always read the label and product information before use.



## METASYS - your strong partner

**METASYS Medizintechnik GmbH**

Florianstraße 3, 6063 Rum bei Innsbruck, Austria

Tel.: +43 512 205420 | Fax: +43 512 205420 7

www.metasys.com | info@metasys.com

GERMANY

+49 8823 938 44 33

info@metasys.com

FRANCE

+33 4 37 90 22 15

info@metasys.fr

ITALY

+39 045 981 4477

desitalia@metasys.com

